

N95 Respirator Mask With 360° Silicone Seal

Code: N95-NIMB-MIW

 **Nimbus**N95

Donning the mask.



Step 1: Perform hand hygiene before applying a mask.



Step 2: Place the respirator over your face with the nose clip at the top, silicone seal facing towards you and the straps in front of the mask.



Step 3: Pull the lower head strap over the head and around the base of your neck.



Step 4: Pull the upper elastic band over the head and place it around the crown of your head.



Step 5: Adjust the respirator by pulling the mask gently down so the silicone seal becomes visible on top of the mask and seals closer to your nose.



Step 6: Adjust the upper and lower head straps to fit your head.



Step 7: Perform a fit test.

Doffing the mask.

Perform hand hygiene. Holding the front of the mask, pull the lower and upper head straps over your head and discard into the bin.

Fit Testing

To fit-test the mask, gently inhale and exhale several times to check for any air leakage. The mask should collapse slightly on inhalation and expand slightly on exhalation. You should feel no air leakage around the mask.

Readjust the fit as required, repeating the donning instructions until you achieve the best fit.

The NimbusN95 is a single use surgical respirator. Never reuse this mask.



Checklist:

- ✓ Mask is firmly positioned under the chin.
- ✓ Mask is fitted across the bridge of nose and cheekbones.
- ✓ Head straps are positioned at the base of neck and on crown of head.
- ✓ Glasses are not fogging.

HygiMed

PROUDLY AUSTRALIAN

www.hygimed.com.au

